

3M Science.
Applied to Life.™



**3M™ Attest™ Steam
Chemical Integrators**
1243A, 1243B, 1243RE

Easy to read results.

Innovative 3M CI technology streamlines your workload.

3M makes it faster and easier to know if surgical instruments and devices have been properly sterilized at the required temperature and time. With 3M™ Attest™ Steam Chemical Integrators, Type 5 you receive the same 3M performance but now with an enhanced, visible design and improved readability.



Easy to see.
Vibrant colors make it easy to locate.



Convenient results.
Moving front technology eliminates time spent interpreting color changes.



Enhanced efficiency.
Simply scan the barcode to upload information to your hospital's instrument tracking system.



Durable material.
Thick aluminum foil is made to withstand the extremes of steam sterilization.

 **3 in 4**

OR professionals agree

3M™ Attest™ Steam Chemical Integrators, Type 5 are visually easier to find in the tray than tested competitive brands.¹

66%

say results are the easiest to read




66% of OR professionals say sterilization results are easiest to read on 3M™ Attest™ Steam Chemical Integrators, Type 5.¹

3M™ Attest™ Steam Chemical Integrator, Type 5 provides a high level of performance.

Steam Chemical Indicator Comparison *	Type 4	Type 5	Type 6
For use in individual load items	✓	✓	✓
Stated values provided on product labeling as recommended in ISO 11140-1:2014	✓	✓	✓
Extensive list of indicated cycles from 250°F to 275°F (121°C to 135°C) (eliminates the need to hold multiple SKUs) see IFU for details	✓	✓	
Performance required to react to ALL critical variables of the sterilization process		✓	✓
Performance correlates to a biological indicator		✓	
Recommended in AAMI ST79 and AORN for use in monitoring loads with an implant along with a BI in a PCD		✓	

*ISO 11140-1:2014 and ANSI/AAMI ST79:2017

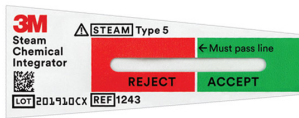



Front

- Easy to read accept/reject result: Moving front ink technology with vibrant “Accept” (Green) and “Reject” (Red) zones for easy interpretation within seconds
- Barcode: Track and record CI lot number and expiration date using your hospital’s current instrument tracking system

Back

- Eye-catching red and white printed back helps make the CI easy to find if flipped in the surgical tray
- Durable aluminum foil is designed to withstand the extremes of steam sterilization temperatures

Ordering information

	Product Name	Catalog Number	Items/ Bag
	3M™ Attest™ Steam Chemical Integrators 2" length	991243A	500
		991243B	100
	3M™ Attest™ Sterilisation Integrator Load Record Cards (not for use within the sterilizer)	991243LR	100

Contact your Pharmacal sales representative at 800-243-5350 for further information.

Distributed By:



Medical Solutions Division
3M Health Care
2510 Conway Ave.
St. Paul, MN 55144 USA

†Gestalt Inc. Customer Quantitative Research Study commissioned by 3M, December 2018. CSSD Managers and OR Nurse Managers showed preference for the following chemical integrators: 3M™ Attest™ Steam Chemical Integrator (76%), 3M™ Comply™ Steri-Gage™ Chemical Integrator (10%), Getinge™ Steam Chemical Integrators (8%), Steris™ Steam Chemical Integrators (7%).

© 2020 3M. All rights reserved. 3M, Attest, Comply and Steri-Gage are marks and/or registered marks of 3M. Unauthorized use prohibited. All other trademarks are property of their respective owners. Please recycle. Printed in USA. EN_GL_70-2011-7921-8